FIL-VA - Monofili sintetici

Via per Schianno, 63, 21100 Varese VA

Made in Italy

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■ ■ Aria+ Class 2500 FFP2 NR

STORAGE AND DISPOSAL

Store and transport unused half masks in their packaging in a dry place [RH<60%] between +5°C and +40°C. Disposable half mask: The filtering half mask should not be used for more than one work shift (8 hours). Do not use after the expiry date indicated on the packaging. The elastic bands have the same manufacturing and expiry date as the complete device. Do not dispose of the device in the environment after use. Dispose of in accordance with local waste management regulations.

WARNINGS

Check the integrity of the half mask before use. Replace the half-mask immediately if difficulty in breathing is experienced or if the half-mask is damaged, or if there are problems with the fit to the face. Beard can reduce the efficiency of the half mask. Do not use when concentrations of contaminants are hazardous to health or life, or in oxygen deficiency. Do not use in an atmosphere where there is a danger of explosion. Maintenance free.

REGULATORY REFERENCES

The CE marking attests that the half mask meets the essential health and safety requirements of Regulation [EU] 2016/425 on personal protective equipment (PPE). The device has been submitted to the certification procedure by the notified body ITEC - ISTITUTO TECNOLOGICO EUROPEO DI CERTIFICAZIONE - Via Biella, 1- 59013 - Montemurlo (PO)-Country: Italy- Notified body number: [2761] which carried out the EU type examination [module B] and performs the conformity to type assessment procedure based on internal production control plus supervised product testing at random intervals [module C2].

DECLARATION OF CONFORMITY

Downloadable declaration of conformity from www.ariapiu.com.

PERFORMANCE RECORDED DURING TECHNICAL TESTS

Total leakage loss towards the inside: less than 1% for at least 46 out of 50 tests carried out; less than 8% for at least 8 out of 10 arithmetic averages for each carrier. Penetration of the filtering material: Test with sodium chloride aerosol 95 l/min <6%; Test with paraffin oil aerosol 95 l/min < 6% Carbon dioxide content in the inhalation air: < 1% Respiratory resistance: Inhalation 30 l/min <0.7mbar; Inhalation 95 l/min < 2.4mbar; Exhalation 160 l/min <3.0mbar. The test results qualify the device as a class FFP2 NR filtering dust half-mask.

Filtering half-mask FFP2 NR without valve Personal protective equipment Category III Non sterile- Single use. Disposable In compliance with EU Reg. 425/2016

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